







GALEAS™/BLADDER Urine Collection Device

Version: V002 Date: Nov 2022

A urine sample collection device used for the collection and transportation of samples for laboratory analysis.

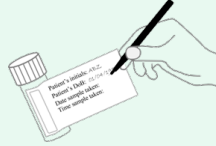


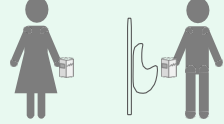
This device contains all necessary materials and instructions for sample self-collection.

	Important: Do not use if damaged or leaking. Keep out of reach of children		Read instructions carefully before using
	Single use only, do not reuse		Research use only
	PRE_GAL_UCD		Temperature limits

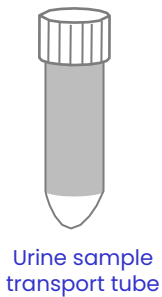
Sample collection instructions: steps 1-10

1. Please ensure the device is within expiry date and read these instructions carefully and completely before attempting to collect the sample.
2. Do not use device if the tube has been damaged or shows signs of leaking.
3. You should not have urinated for at least 2 hours before sample taking.

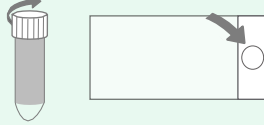
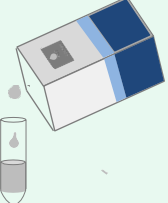
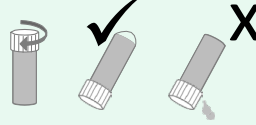

Sample collection instructions

<p>1</p>  <p>Label the urine sample tube with:</p> <ul style="list-style-type: none"> •Your initials •Your date of birth •The date and time that the urine sample is taken 	<p>2</p>  <p>Wash and dry your hands before use.</p>
<p>3</p>  <p>Gently squeeze the sides of the PeeCanter container until the base snaps to lock open. You should not have urinated for at least 2 hours before sample taking.</p>	<p>4</p>  <p>Pass urine into the PeeCanter and place the filled container on a stable surface. It's best to pass urine when you feel you have a full bladder.</p>

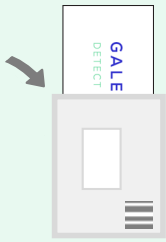
Urine collection device contents



GALEAS Collection device outer box

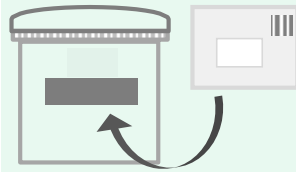
<p>5</p>  <p>Carefully unscrew the cap of the sample tube and leave the cap to one side. Place the tube upright in the holder in the box. Take care not to spill any of the liquid it contains.</p>	<p>6</p>  <p>Pour the urine into the sample tube. Fill the tube to the 50ml fill line displayed on the side of the tube.</p>
<p>7</p>  <p>Screw on the lid of the sample tube securely. Check that there is no leakage. Discard any remaining urine into the toilet and the used PeeCanter in to the regular waste bin.</p>	<p>8</p>  <p>Place the filled tube into the protective clam shell and firmly close. Place the clam shell into the product box & attach the security seal sticker across the opening and bottom of the box.</p>

9



Post the box containing the sample into the pre-addressed postage envelope and seal.

10



Post the samples within 24 hours of collection. The postage has been paid and the package has been designed to fit into any standard post box.

Important checklist

Before you return your sample please do the following:

- Make sure the urine sample tube is labelled and is placed inside the clam shell.
- Place the protective clam shell into the collection device box.
- Close the collection device outer box firmly and apply the return security label to seal it.
- Place the collection device outer box into the return postal envelope.
- Seal the envelope and post within 24 hours. Please store at room temperature until posted.

Safety precautions

Caution should be taken when using the product.

These precautions include, but are not limited to, the following:

1. Avoid contact with liquid in collection tube with eyes, nose and genitals.
2. Do not ingest liquid in collection tube.
3. Do not use device after the expiration date.
4. Once used, do not reuse device.

Material Safety Datasheet (MSDS) available on request.

If additional assistance is required with the product please contact your healthcare provider.